



User's manual

MPS Seat (seat only)

MPS Classic (seat & mobile)

Small & large





WARNING: AVOID SERIOUS INJURY FROM FALLING OR SLIDING OUT.

NEVER LEAVE CHILD UNATTENDED.

ALWAYS KEEP CHILD IN VIEW.

IMPORTANT: KEEP THESE INSTRUCTIONS FOR FUTURE REFERENCE.

A CHILD'S SAFETY IS YOUR RESPONSIBILITY.

ALWAYS USE THE SEAT HARNESS.

SECURE STRAPS TIGHTLY

ALWAYS BE SURE TO LOCK LOCKING CASTERS WHEN NOT MOVING
THE SITTER MOBILE BASE

THIS VEHICLE IS SUITABLE FOR ONE CHILD.

SETTING UP YOUR MPS

UNPACK

Check to see that the following items are included
with your Multi-Positioning Seat (MPS):

- Seat Shell
- Head Cushion installed
- Headrest Tether Strap
- Back Cushion installed
- Seat Cushion (Regular or Extended size) installed
- Two Safety Attachment Straps installed
- Five Point Harness System installed
- Top Tether Strap
- Left and Right Leg Rest Extensions
- 2 Leg Rest GROOVED Knobs (1 3/16")
- 2 Footrest BALL Knobs (1 1/2")

- Note: Make sure the
chair is stable!




ASSEMBLY

First attach the MPS to the appropriate base of choice with the Attachment Straps:



MOBILE FLOOR BASE

- (A) Secure tightly the Rear Strap under lip of Mobile Floor Base. Secure tightly the Bottom Strap under Mobile Floor Base between both sets of wheels

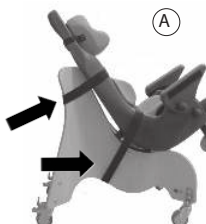


PUSH CHAIR SEAT

- (B) Secure tightly the Rear Strap around back of Push Chair Base, between the fabric and the cross bars.
- (C) Secure tightly the Strap under Push Chair Base under the seat making sure to thread between the seat back frame and the front leg frame.

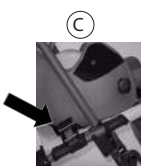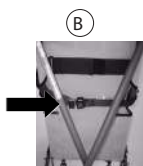

STANDARD CHAIR

- (D) Secure tightly the Rear Strap around back of chair (spiral method below or straight method in Figure 1.) Secure tightly the Bottom Strap under seat of chair Figure 3. (D)

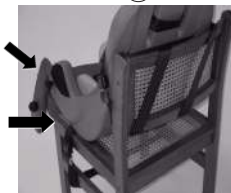


FOOTREST ASSEMBLY

Your Special Tomato® Multi-Positioning Seat (MPS) comes partially assembled for your convenience.

- Insert both ends of the Leg Rest Extensions into the corresponding holes in the front of the seat shell.
- Make sure to insert the leg rests far enough so that the Leg Rest GROOVED Knobs (1 3/16") go through one of the three corresponding holes in the leg rests, giving the appropriate depth for your child (A)
- Tighten the 2 Leg Rest GROOVED Knobs (1 3/16"). Overtightening is not necessary! Turn it until you feel resistance. There is a block to prevent tightening it too much.
- Place the footrest at the appropriate height, depth, and angle for your child.
- Insert the 2 Footrest BALL Knobs (1 1/2") into one of the three appropriate holes on the side of the footrest. These Footrest BALL Knobs (1 1/2") should be tight! (B)

Please Note: It is NOT necessary to overtighten any of the knobs. They are designed to hold with a SNUG fit!

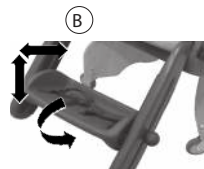

ADJUSTMENTS

The Special Tomato® Multi-Positioning Seat provides a comfortable and adjustable seating surface for your child. The durability of this unique product along with built-in adjustability allow for years of use as it grows with your child. Adjustability is accomplished as follows:


HEAD CUSHION & BACK CUSHION

Adjust up and down easily. Hook and loop strips and our track reference mark system allow you to quickly reposition the modular cushions to fit your child.




SEAT CUSHION

Uses hook and loop strips plus our track system for easy adjustment. Under the Seat Cushion are three guide lines.

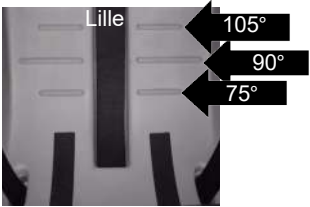

The middle line positions the hips at 90°. The top line provides 15° of Hip Extension. The bottom line provides 15° of Hip Flexion. Our Hip Flex® Adjustment provides 15° of Hip Flexion or 15° of Hip Extension without recline.

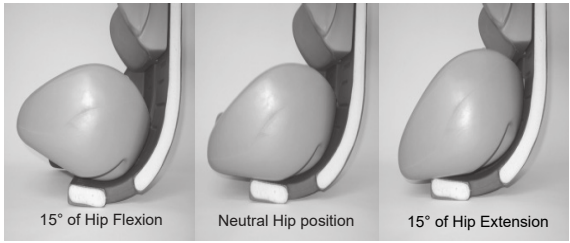

FOOT REST

Provides maximum adjustment allowing for height, depth, and angle adjustability. Hook and loop straps. Secure your child's feet to the footrest for proper positioning.



Adjust the depth of the footstraps as needed.



FIVE-POINT HARNESS



Provides comfort and adjustment needed to properly support and position your child. A soft built in chest pad promotes comfort and provides support by simply tightening the over the shoulder straps as needed.



To adjust the Five-Point Harness, remove the black covers from the shoulder straps. Press in the gray button.




Pull the top strap to tighten.



Pull the bottom strap to loosen.

LAP STRAP

Press the center red button to release both sides of the lap strap. Re-insert each metal buckle into the corresponding socket to secure the lap strap portion of the 5-point harness.

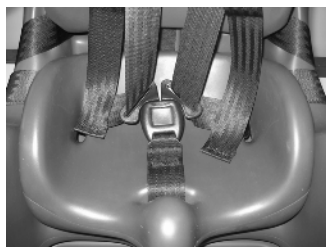

MOBILE FLOOR BASE

- (A) Complete with four swivel casters.
- (B) Rear casters have foot-operated locks.
- (C) Tray Storage in rear of Mobile Floor Base. Stay secure with hook placements.



OPTIONAL SEAT CUSHIONS

Extended Seat Cushion can be purchased as the standard seat cushion with your MPS depending on your child's dimensions.

Extended Seat Cushion can be purchased at a later date as an optional accessory if your child needed the Regular Seat Cushion at the time of your original purchase. This way you can extend the "life" of the MPS without having to buy a completely new seating system.



SWITCHING SEAT CUSHIONS



Unclip crotch strap and pull down through hole in seat cushion.
Remove seat cushion.



Verify that the 5-Point Harness Clip is securely fastened.
Push small metal clip through slot of large metal clip to secure the 5-Point Harness.



Test it by pulling on both straps to make sure that they stay connected



Place new cushion in seat shell.

Pull crotch strap back up through seat cushion.

TRAY

Tray items include:


Tray

2 Tray Risers

2 Tray Riser BALL Knobs (1 1/2")

2 Washers

- (A) Secure both Tray Risers to the Seat Shell at desired angle (0° - 30°) with the attached hook and loop strips.
- (B) Place Washers on BALL Knobs (1 1/2") and insert them into the corresponding slot and hole. Tighten both BALL Knobs (1 1/2").(Not necessary to over tighten.)
- (C) Place the Tray on the Tray Risers with the attached hook and loop strips on the bottom of the Tray surface considering the appropriate amount of depth for your child. Use the hook and loop flap to secure the Tray to the Tray Risers.
- (D) To remove the tray, first pull up on the hook and loop flap.
- (E) Second, pull the tray off by holding onto the curved sides and rotating one side towards you.
- (F) The Tray Risers can remain on the Seat Shell.

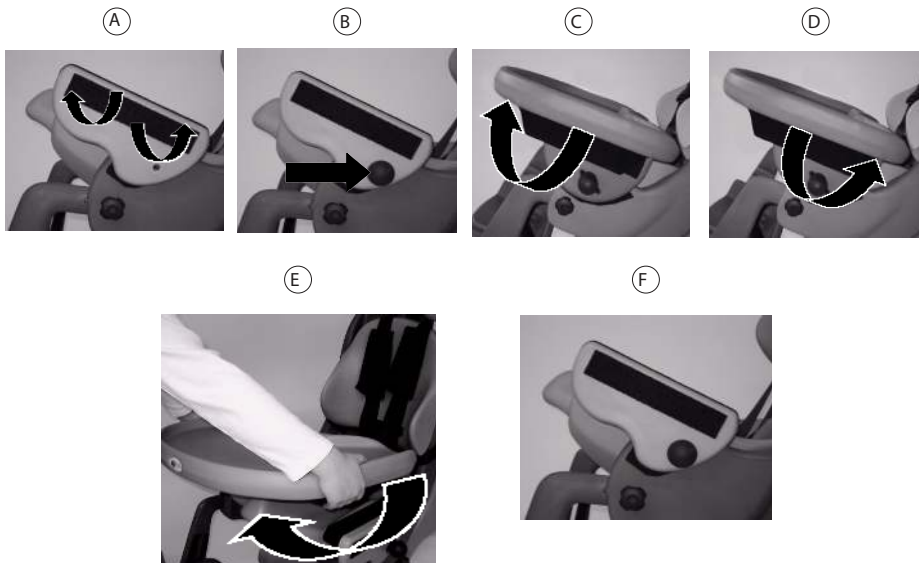

PUSH CHAIR BASE

If you have purchased the MPS Push Chair Base, you will receive a separate manual with your MPS Push Chair Base.

LIMITED WARRANTY

Bergeron by Design warrants to the original retail purchaser of the Special Tomato® product, that if any part or component proves defective in material or workmanship within two/five years of the purchase date, the defective part will be repaired or replaced (at Bergeron by Design's discretion) free of charge. Warranty service may be performed by an authorized service center or (at Bergeron by Design's discretion) the factory.

This warranty does not cover normal wear and tear, or damage caused by accident or misuse.

To exercise this limited warranty, the user should first obtain a Return Authorization Number from the dealer the Special Tomato® product was purchased from. The product must be delivered charges pre-paid (UPS recommended) to the factory or to an authorized service center, together with a copy of the original invoice, the Return Authorization Number and a written description of the problem

MAINTENANCE & PRODUCT CARE

Clean chair with damp cloth; then wipe dry. With normal use, over time screws may loosen. Please maintain tight screws. Use chair on smooth level surface.

HARNESS CARE

Place harness in a mesh bag and machine wash with cold water, then air dry.

CUSTOMER SERVICE

Contact your dealer directly. Or contact the European Authorized Representative.

NOTICE

No part of this document may be photocopied, reproduced, transmitted, transcribed, stored in a retrieval system or translated to another language or computer language, in any form or by any means, electronic, mechanical, magnetic, optical, chemical, manual, or otherwise without the prior written consent of Bergeron by Design.

Use only Special Tomato® accessories and parts on Special Tomato® products. Special Tomato® parts are not interchangeable with other manufactures' products. Replace any worn parts immediately for safety. © Bergeron By Design



IMPORTANT- WARRANTY WILL BE VOID IF DEFACED OR REMOVED

Your MPS is guaranteed to be free from any manufacturing defects for a period of 2/5 years from the date of purchase under normal use and provided it is used in compliance with the operating instructions. This warranty extends only to the original retail purchaser and is only valid when supplied with proof of purchase. Please retain proof of purchase with this Limited Warranty.

Should a repair be needed please contact the retailer from whom the product was purchased.

PLEASE NOTE THAT THE WARRANTY WILL NOT BE APPLICABLE IF:

- A fault is the cause of misuse or poor maintenance.
- Repairs are carried out by a third party.
- The stroller is faulty due to general wear and tear which is the result of everyday use.
- A stroller is damaged as a result of an accident.
- The manufacturer is not satisfied that the stated warranty terms and conditions have been met.



MANUFACTURED BY:
BERGERON HEALTH CARE
15 SOUTH SECOND STREET
DOLGEVILLE, NY 13329

PRODUCT CODE:

PRODUCT DESCRIPTION:

COUNTRY OF ORIGIN:

**AUTHORISED
EUROPEAN**

REPRESENTATIVE:

GLOBAL CAREHAB
Holmstrupgaardvej 11
8220 Brabrand
Denmark

Tel: +45 24 65 55 50

Web: www.globalcarehab.com

Date:

Id:



Bergeron By Design
15 South Second Street
Dolgeville, New York 13329
T 1 315 429 8407
F 1 315 429 8862

© 2009 Bergeron By Design
The information contained herein is subject to change without notice.
Bergeron By Design shall not be liable for technical errors or omissions herein.