


# Sitter Harness

## SITTER HARNESS THREADING & CONNECTIONS

Harness diagram

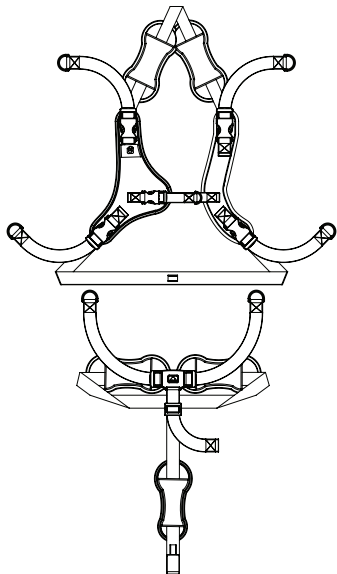

Harness installed




Chest straps  
5-point harness

3-point pelvic harness  
Waist & crotch straps

Attachment strap  
To attach to a chair, etc.



## PELVIC HARNESS INSTALLATION

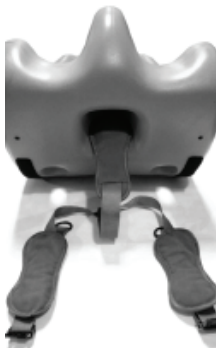

Step 1  
Crotch strap up through  
bottom center slot



Step 2  
Right waist strap up  
through right slot



Step 3  
Left waist strap up through  
left slot



Step 4  
Clip buckling

# CHEST HARNESS INSTALLATION



**Step 1**  
Unclip right chest strap  
and slide strap through left  
shoulder slot



**Step 2**  
Thread strapping around  
the back and out the right  
shoulder slot



**Step 3**  
Clip to the top of the right  
shoulder chest harness



**Step 4**  
Unclip right lateral strap  
and slide strap through left  
waist slot



**Step 5**  
Tread strapping under  
through crotch loop and up  
and out the right slot



**Step 6**  
Reattach the chest clip to  
the bottom of the right  
side of the chest harness



**Step 1**  
Tread strap down through  
the left waist slot



**Step 2**  
Tread strap under sitter on  
top of loop and up right  
waist slot

## ATTACHMENT STRAP INSTALLATION



**MANUFACTURED BY:**  
BERGERON HEALTH CARE  
15 SOUTH SECOND STREET  
DOLGEVILLE, NY 13329  
**PRODUCT CODE:**

**PRODUCT DESCRIPTION:**

**COUNTRY OF ORIGIN:**

Bergeron By Design  
15 South Second Street  
Dolgeville, New York 13329  
T 1 315 429 8407  
F 1 315 429 8862

**AUTHORISED  
EUROPEAN  
REPRESENTATIVE:**

GLOBAL CARE AD  
ROSENWANGS ALLE 32  
81263 WILLY J  
DENMARK  
TEL +45 24 66 55 60  
WEB [www.globalcaread.com](http://www.globalcaread.com)



Date: d.



© 2009 Bergeron By Design

The information contained herein is subject to change without notice.  
Bergeron By Design shall not be liable for technical errors or omissions herein.