



User's manual Sitter Seat





WARNING: AVOID SERIOUS INJURY FROM FALLING OR SLIDING OUT.
 NEVER LEAVE CHILD UNATTENDED.
 ALWAYS KEEP CHILD IN VIEW.
 IMPORTANT: KEEP THESE INSTRUCTIONS FOR FUTURE REFERENCE.
 A CHILD'S SAFETY IS YOUR RESPONSIBILITY.
 ALWAYS USE THE SEAT HARNESS.
 SECURE STRAPS TIGHTLY.
 ALWAYS BE SURE TO LOCK LOCKING CASTERS WHEN NOT MOVING
 THE SITTER MOBILE BASE.
 PREVENT SERIOUS INJURY OR DEATH - DO NOT USE IN MOTOR VEHICLES.

PARTS

- 1 Sitter with bolted attachment strap
- 1 chest strap
- 1 crotch strap
- 1 waist strap
- 1 secondary attachment strap

Universal
attachment straps



5 WAY TO USE YOUR SPECIAL TOMATO® SITTER

*Use universal attachment straps to secure sitter in below applications



On Chair



Stationary Base



Mobile Base




Floor Sitter



In Push Chair

SITTER HARNESS THREADING & CONNECTIONS

Harness diagram

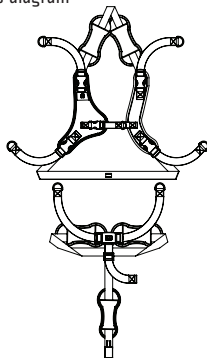

Harness installed



Chest straps
5-point harness

3-point pelvic harness
Waist & crotch straps

Attachment strap
To attach to a chair, etc.



PELVIC HARNESS INSTALLATION



Step 1
Crotch strap up through
bottom center slot



Step 2
Right waist strap up
through right slot



Step 3
Left waist strap up through
left slot



Step 4
Clip buckling

CHEST HARNESS INSTALLATION



Step 1
Unclip right chest strap
and slide strap through left
shoulder slot




Step 2
Thread strapping around
the back and out the right
shoulder slot



Step 3
Clip to the top of the right
shoulder chest harness



Step 4
Unclip right lateral strap
and slide strap through left
waist slot

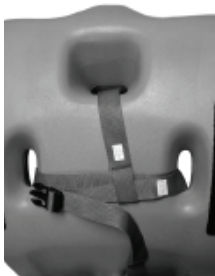

Step 5
Tread strapping under
through crotch loop and up
and out the right slot



Step 6
Reattach the chest clip to
the bottom of the right
side of the chest harness



Step 1
Tread strap down through
the left waist slot




Step 2
Tread strap under sitter on
top of loop and up right
waist slot


ATTACHMENT STRAP INSTALLATION

ATTACHMENT STRAPS & APPLICATIONS


Attached to a chair

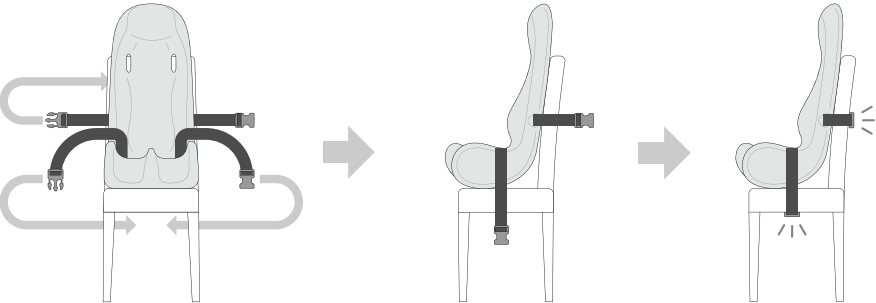


Attached to a Special Tomato Wooden Base

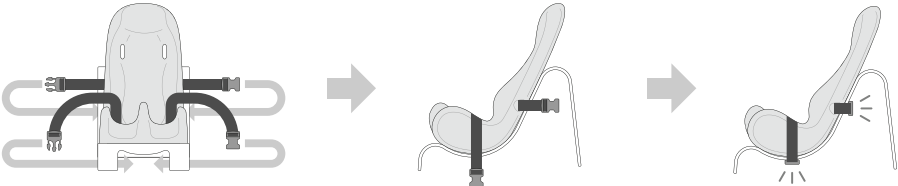

Attached to a Special Tomato floor Base

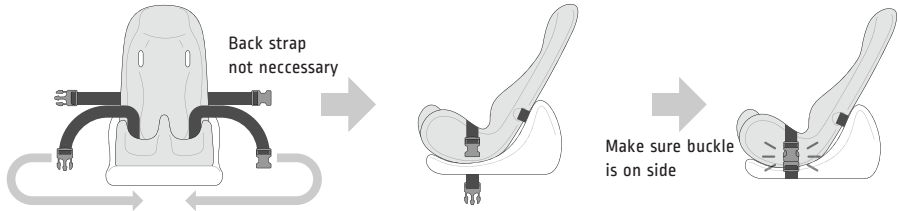

Attached to a Special Tomato Jogger, EIO or Push Chair

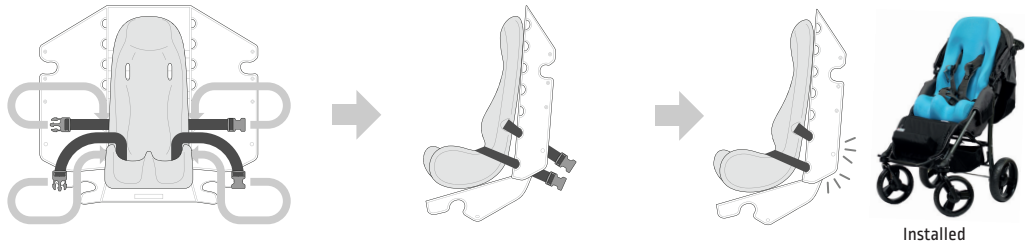

SITTER HARNESS FITTING
Proper adjustment techniques



Use the shoulder covers to prevent your child's neck and face from rubbing on the plastic buckles

To ensure proper placement of the chest harness on your child, make sure the chest strap lays in the middle of the chest (nipple level).

The pelvic harness has a cam lock which prevents the pelvic belt from loosening to maintain proper positioning of the pelvis.

The pelvic belt has one quick-release button that unlocks both sides of the pelvic harness for quick and easy access to your child.

Adjust the length of the crotch strap to allow the pelvic belt to lay across the pelvis. This will prevent the pelvic belt from laying across the lower abdomen.



The pelvic belt has a cam lock to keep it at the desired length. for growth adjustments, push up on webbing to release the cam lock.

25° TILT POSITION
All bases can tilt up to 25°



Step 1
Unclip attachment straps

Step 2
Adjust to desired angle

Step 3
Clip attachment straps to secure

LIMITED WARRANTY

Bergeron by Design warrants to the original retail purchaser of the Special Tomato® product, that if any part or component proves defective in material or workmanship within five years of the purchase date, the defective part will be repaired or replaced [at Bergeron by Design's discretion] free of charge. Warranty service may be performed by an authorized service center or [at Bergeron by Design's discretion] the factory.

This warranty does not cover normal wear and tear, or damage caused by accident or misuse.

To exercise this limited warranty, the user should first obtain a Return Authorization Number from the dealer the Special Tomato® product was purchased from. The product must be delivered charges pre-paid [UPS recommended] to the factory or to an authorized service center, together with a copy of the original invoice, the Return Authorization Number and a written description of the problem

MAINTENANCE & PRODUCT CARE

Special Tomato® Products are impermeable to fluids, but do not submerge. Use a mild detergent/cleaner to wipe the chair periodically.

To extend the life of the hook and loop strips, comb out the hook portion [hard side] to remove lint and other debris.

HARNESS CARE:

Place harness in a mesh bag and machine wash with cold water, then air dry.

CUSTOMER SERVICE

Contact your dealer directly. Or contact the European Authorized Representative.

NOTICE

No part of this document may be photocopied, reproduced, transmitted, transcribed, stored in a retrieval system or translated to another language or computer language, in any form or by any means, electronic, mechanical, magnetic, optical, chemical, manual, or otherwise without the prior written consent of Bergeron by Design.

Use only Special Tomato® accessories and parts on Special Tomato® products. Special Tomato® parts are not interchangeable with other manufactures' products. Replace any worn parts immediately for safety. © Bergeron By Design



IMPORTANT- WARRANTY WILL BE VOID IF DEFACED OR REMOVED

Your Sitter Seat is guaranteed to be free from any manufacturing defects for a period of 5 years from the date of purchase under normal use and provided it is used in compliance with the operating instructions. This warranty extends only to the original retail purchaser and is only valid when supplied with proof of purchase. Please retain proof of purchase with this Limited Warranty.

Should a repair be needed please contact the retailer from whom the product was purchased.

PLEASE NOTE THAT THE WARRANTY WILL NOT BE APPLICABLE IF:

- A fault is the cause of misuse or poor maintenance.
- Repairs are carried out by a third party.
- The stroller is faulty due to general wear and tear which is the result of everyday use.
- A stroller is damaged as a result of an accident.
- The manufacturer is not satisfied that the stated warranty terms and conditions have been met.



MANUFACTURED BY:
BERGERON HEALTH CARE
15 SOUTH SECOND STREET
DOLGEVILLE, NY 13329

PRODUCT CODE:

PRODUCT DESCRIPTION:

COUNTRY OF ORIGIN:
USA

**AUTHORISED
EUROPEAN
REPRESENTATIVE:**



GLOBAL CARE AB
ROSENVANGS ALLE 32
8260 VIRY J
DENMARK
TEL +45 24 66 50 50
WEB www.globalcareab.com

Bergeron By Design
15 South Second Street
Dolgeville, New York 13329
T 1 315 429 8407
F 1 315 429 8862



© 2009 Bergeron By Design

The information contained herein is subject to change without notice.
Bergeron By Design shall not be liable for technical errors or omissions herein.